

SECOND DEGREE BURN TREATED WITH PRINCIPELLE IF OINTMENT, MELMAX AND PRINCIPELLE MATRIX

t=0

t=37 days



01-02
Male (age 35)
presents second
degree burn.

Treatment with
Flamazine and fatty
gauze is not
satisfactory.



07-02
Treatment with
antibacterial
Principelle IF
Ointment and
MelMax 08x10 cm,
covered with
Principelle Matrix to
absorb excess
wound exudate.



19-02
Patient shows
progressive
improvement after
12 days (2 dressing
changes).

Wound is closed
and skin is in
remodeling stage.



10-03
Skin was kept
flexible with a third-
party skin cream to
allow for smooth
recovery.

Wound fully healed
without loss of
function, side effects
or scarring.

Product used:
Principelle IF
Ointment
(PPIF15)



Product used:
MelMax (MM081010)
Sterile honey impregnated
wound dressing.



Product used:
Principelle Matrix (PPM1010)
Sterile impregnated
wound dressing

