## ChitoClot Gauze

**EN** Instructions for AnsCare ChitoClot Gauze

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**TU** İçin talimatlar AnsCare ChitoClot Gazli Bez

### Description

AnsCare ChitoClot Gauze is a 100% Pure Chitosan Hemostatic Dressing intended to promote wound bleeding-control and to provide a wet-surrounding for wound care purpose.

### Intended Use

1. AnsCare ChitoClot Gauze is indicated for use of bleeding wound management. It promotes rapid control of moderate to severe bleeding and exudates absorption.
2. It is indicated for the following wounds, including lacerations, abrasions, puncture wounds, and contusions procedures.

### Instruction for Use

1. Use appropriate size and number of ChitoClot Gauze based on the wound area and amount of bleeding or exudates.
2. Clean the wound surrounding.
3. Apply ChitoClot Gauze directly to the wound.
4. To control bleeding, apply firm pressure for 3 minutes. Secure it with adequate cover dressing, maintaining pressure to the wound.
5. When remove or replacement, use sterile saline or water to rinse the gauze and any remnants.

### Warnings and precautions

1. Read the instructions carefully before using the device. Do not use the product beyond its expiration date.
2. Inspect the product and its outer package for damage before use. Do not use if the product or its package is damaged or incomplete.
3. The device should be used to infected wounds under the supervision of the medical staff.
4. This product is for single use only. Reuse is prohibited and may cause the risk of infection to the patient or the user.
5. This product is for external use only. Do not apply it inside the body.
6. The disposal of the subject device is following the rule of the Official Agency.
7. The device shall be removed within 24 hours.
8. Contains chitosan from crab, if you have any concerns with shellfish allergy, please contact medical professionals.

### Sterilization and conservation

1. Gamma radiation sterilization has been employed. Do not resterilize.
2. Store at a dry condition and room temperature (15~35°C). Avoid direct sunlight, high temperature and humid environment.

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