# ChitoClot Bandage

- EN** Instructions for Anscare ChitoClot Bandage
- DE** Anweisungen für Anscare ChitoClot-Bandage
- SP** Instrucciones para Vendaje Anscare ChitoClot
- PT** Instruções para Anscare ChitoClot Bandage
- FR** Instructions pour Bandage ChitoClot Anscare
- IT** Istruzioni d'uso Bendaggio Anscare ChitoClot
- NL** Gebruiksaanwijzing voor Anscare ChitoClot Verband
- TU** İçin talimatlar Anscare ChitoClot Bandaj

## EN English

### Description

AnsCare ChitoClot Bandage is a topical wound dressing intended to promote wound bleeding-control and to provide a wet-surrounding for wound care purpose.

### Material

AnsCare ChitoClot Bandage is made from 100% chitosan non-woven material with an adhesive back sheet.

### Intended Use

1. AnsCare ChitoClot Bandage is indicated for use of bleeding control. It promotes rapid control of wound bleeding and exudates absorption. In addition, it acts as a protective barrier to prevent wound from abrasion, friction, and contamination.
2. It is indicated for the following wounds, including lacerations, abrasions, puncture wounds, and opening contusions.

### Instruction for Use

1. Clean the wound surrounding.
2. Remove the release liner from the back of bandage, and ensure the chitosan layer in the center of bandage against the wound.
3. Apply ChitoClot Bandage directly to the wound, and then fix it with its adhesive back sheet with pressure.
4. The bandage may be removed directly, or it may be removed after being rinsed by normal saline or water.
5. The bandage is recommended to remove within 24 hours.

### Warnings and precautions

1. Read the instructions carefully before using the device. Do not use the product beyond its expiration date.
2. Inspect the product and its outer package for damage before use. Do not use if the product or its package is damaged or incomplete.
3. The device should be used to infected wounds under the supervision of the medical staff.
4. Post-operative care should be accomplished
5. If the patient experiences discomfort during or after the procedure, medical advice should be sought immediately to provide the appropriate treatment.
6. This product is for external use only. Do not apply it inside the body or on mucous membrane.
7. The disposal of the subject device is following the rule of the Official Agency.
8. Contains chitosan from crab, if you have any concerns with shellfish allergy, please contact medical professionals.
9. This product is for single use only. Reuse is prohibited and may cause the risk of infection to the patient or the user.

### Sterilization and conservation

1. Gamma radiation sterilization has been employed. Do not resterilize.
2. Store at a dry condition and room temperature (15-35°C). Avoid direct sunlight, high temperature and humid environment.

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**CE** **EC REP** Obelis S.A
BenQ Materials Corporation (A BenQ Group Company)