Ansca+e LeniScar Silicone Stick provides an easy-to-use topical scar treatment. It is intended to diminish hypertrophic scars and keloids characterized with a raised or discolored appearance. LeniScar is an advanced silicone formula which contains medical grade silicone and antioxidant vitamin C & E. It creates a water-proof, transparent thin film which helps to maintain the moisture balance of skin and provides an optimal protective barrier for the scar to restore.

Product Name

Ansca+e LeniScar Silicone Stick

Intended user Lay person/Medical professional person

Indications Prevention of hypertrophic scars and keloids resulting from surgery

injury, C-section, burns, cuts, scrapes, acne, insect bites, etc.

Ingredients Poly(dimethylsiloxane), Silicone elastomer, Petroleum wax, Ascorbyl

tetra 2-hexyldecanoate, Tocopherol acetate.

Directions

1. Ensure scar area is clean and dry before each application.
2. Apply LeniScar Silicone Stick over the entire scar toward the same direction.
3. Repeat 2-3 times daily for consecutive 8-12 weeks or until scar stops responding.
4. To remove, simply wash with a mild soap or neutral cleanser.

Precautions


1. For external use only.
2. Do not use on open wounds or unhealed skin.
3. Avoid contact with eyes and mucous membranes.
4. If irritation or allergic reaction occurs, discontinue use and consult your physician.

Tips


SP-200-1 (9.2g) ----- SP-200-2 (4.0g) -----

Large Scar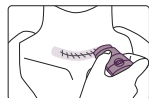

Surgery Injury

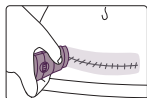
C-section

Minor Scar

Storage Store at 15-35°C and avoid from direct sunlight, heat and moisture.

Adverse Incident Reporting Please report to BenQ Materials Corporation, our Authorized Representative, and the competent authority of the Member State in which the user and/or patient is established if any serious incident that has occurred in relation to the device.

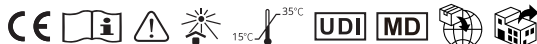CMC Medical Devices & Drugs S.L.
C/ Horacio Lengua n.º 18, CP 29006, Málaga, Spain.
info@cmcmmedicaldevices.com

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